



**Lower Back Pain
Pressure Ulcers-Sacral Area
Incontinence**

Place 5"x 8" emitter pads anterior and posterior about 1" to 2" apart on both sides of the buttocks and abdomen, or large 8"x 10" emitter pads directly over the lower back and over the lower abdomen. For treatment of extensive wounds in sacral area: remove existing dressing and cover wound with a sterile gauze pad soaked in sterile saline solution. Apply emitter pad directly over wound. Place second emitter pad on the lower abdomen of patient. At completion of treatment, redress wound with the product of your choice.